# **Case Report Series**

Gap Management and Placement of a **CONTACT Implant in the Esthetic Zone** 





Dr. Konrad Meyenberg, **Switzerland** First published in: Restorative **Gap Management in the Esthetic** Zone, Konrad Meyenberg, **Quintessence Publishing** 



## The Baseline Situation

#### Patient with root resorption due to ankylosis

A patient presented at the dental office with root resorption due to an ankylosed maxillary right central incisor with an extended crown. The tooth had been damaged during an accident a few years before. A gap was present between the maxillary right canine and right lateral incisor.





#### Extraction, ridge reconstruction, and provisional

Extraction and immediate reconstruction of the ridge with guided bone regeneration and a soft tissue graft

Provisional RPD during the first 6-month healing phase to enable a slight distalization of the right lateral incisor with an internal spring







## **Implantation**

After 6 months, placement of a CONTACT implant PF 4.5\*

Simultaneous soft tissue grafting

Conditioning of augmented soft tissue with an individualized healing





The Treatment Strategy



#### **Prosthetic restoration**

Implant impression taking after soft tissue conditioning

Preparation of screw-retained glass ceramic crown cemented on individualized zirconia abutment as final restoration\*\*

Renewal of the composite buildup on the left central incisor









### The Outcome

The performed gap closure combined with the placement of a CONTACT implant led to convincing esthetics after finishing the treatment. An excellent mechanical stability of the zirconia abutment walls was possible due to the small abutment screw diameter





