

DRILLING PROTOCOL FOR CONTACT

PF \varnothing 3.5 endosseous \varnothing 2.7	PF \varnothing 4.0 endosseous \varnothing 3.5	PF \varnothing 4.5 endosseous \varnothing 3.5	PF \varnothing 5.0 endosseous \varnothing 4.2	PF \varnothing 6.0 endosseous \varnothing 5.0	Rotational speed rpm
2.0 Pilot drill	2.0 Pilot drill	2.0 Pilot drill	2.0 Pilot drill	2.0 Pilot drill	800
↓	2.8 VECTOdrill	2.8 VECTOdrill	2.8 VECTOdrill	2.8 VECTOdrill	600
	↓	↓	3.5 VECTOdrill	3.5 VECTOdrill	500
			4.3 VECTOdrill	400	
2.7/3.5 CONTACT Profile drill	3.5/4.0 CONTACT Profile drill	3.5/4.5 CONTACT Profile drill	4.2/5.0 CONTACT Profile drill	5.0/6.0 CONTACT Profile drill	250-300

