


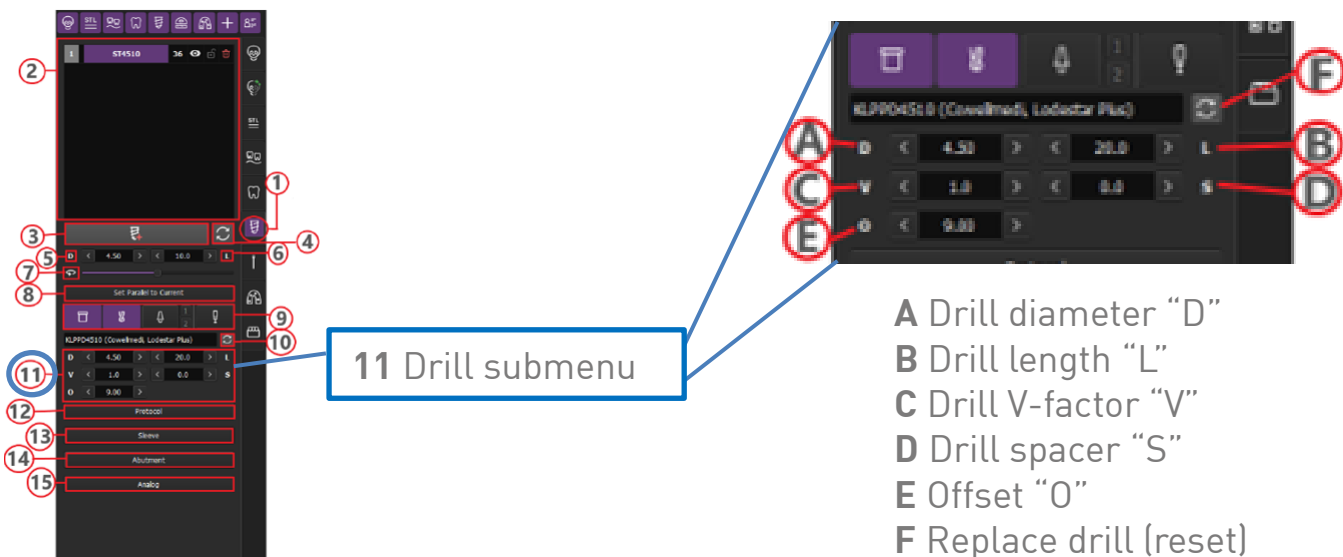
# Working in the ImplaStation Planning Software

## Important Information for Thommen Medical Users

When working with Thommen Medical products in the ImplaStation planning software, please make sure to carefully read the official instruction manual for the software before starting. The manual can be found on the website of the software manufacturer.

**Important:** The planning software ImplaStation allows manual adjustments to critical drilling parameters such as the V-Factor, drill offset, drill length and spacer (see screenshots of the menu below).

 These values are strictly defined and pre-set for the Thommen Medical implant system and **must not be changed. Adjusting them may lead to incorrect implant site preparation and compromise clinical safety.**



If any of the above-named settings have been changed by mistake, please contact ImplaStation technical support directly for assistance: [info@prodigident.com](mailto:info@prodigident.com)

For all questions regarding the correct use of our implant system or our guided surgery solutions in general, feel free to visit our website [www.thommenmedical.com](http://www.thommenmedical.com) or contact us via [info@thommenmedical.com](mailto:info@thommenmedical.com).